




## Helpful Tips for Using Your CapTel Phone

### Adjusting the Caption Font Size and Color

Did you know that you can adjust the way the captions appear on your CapTel phone? You can change the font size and color to a setting that is easiest for you to read. Go ahead and experiment with these options to find what works best for you!

 **Note:** changing the font size and color applies only to conversation captions. The font and color for menu settings always remain the same.

#### To change the captions font size and color:

##### CapTel 840/840i

In the Options menu, select Settings then select Display Settings. From this screen, you can set the Conversation Colors or set the Conversation Font Size. **Note:** you cannot change the font size or color during a live call.




[Larger view](#) »

 [840i: Changing the Caption Font Size & Color](#)




##### CapTel 2400i

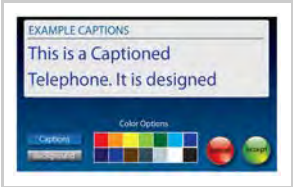
You can choose from different font styles as well as multiple sizes and colors for the captions.




[Larger view](#) »

#### To change the captions font size and color:

From the idle screen touch,  
 **Settings** →  **Display** →  **Font Settings**

 [2400i: Adjusting the Font Size](#)  
[2400i: Adjusting the Font Color](#)

 **Tip:** With the CapTel 2400i, you can adjust the font size during a live call. Just touch + or – on the display screen to make the font larger or smaller.

## Protecting Your Hearing in the Great Outdoors

From mowing the lawn to ball games to concerts and fireworks, spending time outside can be noisy! With the official start of summer, now is the perfect time to review some tips for protecting your hearing outdoors.

 [Read more.](#)

## CapTel in the Community


Be on the lookout for CapTel this summer at upcoming conferences and events. CapTel is proud to sponsor several hearing loss related events, including the HLAA conference in Salt Lake City and the upcoming TDI conference in Maryland. Please stop by and say hi!



Have a CapTel story to share? Send it to [Newsletter@CapTel.com](mailto:Newsletter@CapTel.com). We'd love to hear from you!

**CapTel.com | CapTel Support | CapTel News & Blog**  
CapTel Inc. 450 Science Drive, Madison, WI 53711  
©2017 Ultratec, Inc.

CapTel is a registered trademark of Ultratec, Inc.  
All trademarks used here are property of their respective owners.

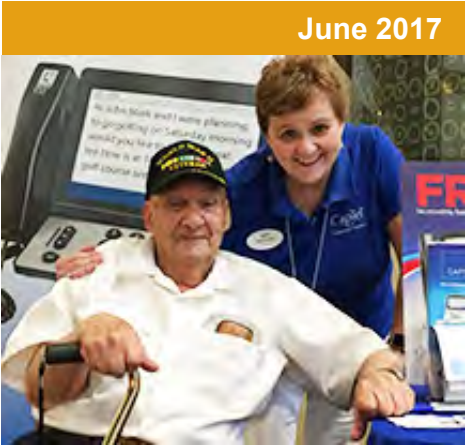









## 'Round the Clock Help Just one button away!

Help is available 24 hours a day, 7 days a week!\*

*\*Excluding major holidays.*

To contact:

-  Just press the blue button on your CapTel phone!
-  1-888-269-7477  
1-866-670-9134 (Spanish)
-  [Online Chat](#)
-  [CapTel@CapTel.com](mailto:CapTel@CapTel.com)

Like us on   
Follow us on   
Watch us on   
Visit us on 

Thank you to everyone who follows CapTel on Facebook. With more than 21,000 followers, the online community of CapTel users is another great resource.