







CapTel 2400i: Customizing the Tone Setting



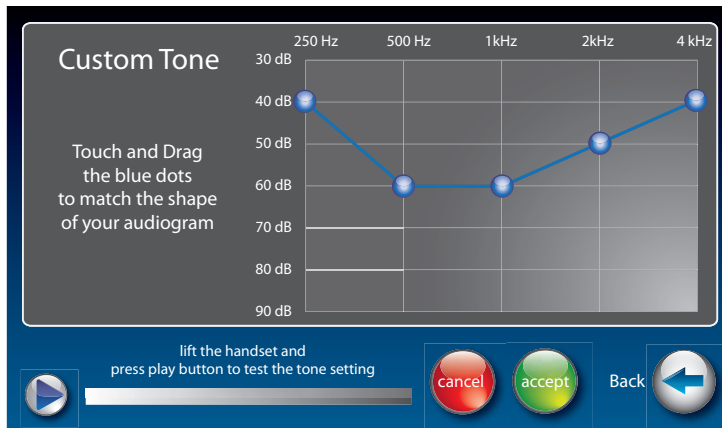

CapTel 2400iBT lets you personalize the Tone setting to enhance the frequency ranges that you hear best. For example, you can adjust the tone settings to match your own personal audiogram. CapTel lets you listen to test each setting as you make changes, to find the optimal customized hearing level.

To customize the Tone setting:

1. On the display screen, touch  **(Settings)**.
2. Touch  **(Sounds)**.
3. Touch  **(Tone)**.
4. On the display screen, touch  **(Custom)**.
5. Touch and drag the blue circles on the frequency chart to adjust the various sound levels.

As you make changes, test the sound quality by touching  **(play)** while listening over the handset.

TIP *If you have a personal audiogram, adjust the blue circles to match the shape of your audiogram to give you optimal sound levels for your personal hearing level.*

6. When you are finished, hang up the handset and touch  **(Accept)** to save your changes or  **(Cancel)** to exit.